

Hot Tips Report

ID	0015	Description	TN 06.16 - Driver for Socket CF Card Rev 'E'
Date	4/26/2005 9:30:26 AM		
Manager	ADMIN	Status	Active

Text **Driver for Socket CF Card Rev 'E'**

The information in this Tech Note applies to:

- hp iPAQ with Pocket Medic using a Socket CF card adapter.

Problem

A communication problem has been reported when using the hp iPAQ with the Socket Serial I/O CF Card Adapter revision 'E'. This problem has been reported by hp iPAQ 2210 users, however it may occur with additional hp iPAQ models.

To determine which version of the Serial I/O CF Card you have, please find the part number and revision level of your card on the label on the back of the card. For example, a socket card with XXXX-XXXXX A* on the back is at revision 'A'.

Solution

1. Close Pocket Medic by selecting 'File' and 'Exit'.
 2. Download the driver for the Socket Serial I/O CF Card Adapter from the following link: http://www.socketcom.com/support/support_sio.asp?Type=SIO
 3. Run ActiveSync and ensure communication has been established. The icon should be solid green.
 4. Run the install file downloaded from the Socket website.
 5. Insert the Socket Serial I/O CF Card Adapter.
-